


MEDIWISS Analytic GmbH

Uerdinger Strasse 3
D-47441 Moers
02841/8890470
pwahl@mediwiss-analytic.de

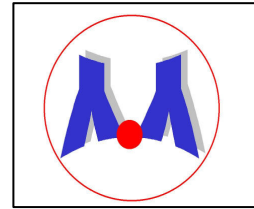

AllergyScreen™/AlleisaScreen™ CIRCULAR LETTER 9

Saving data

Sometimes it can be necessary to deinstall the complete RapidReader software. If this happens and you want to save the former produced measurements of the database, please check first, which software version you have installed.

Case 1:

**The old version is 500 or higher (510/ 520/ 530/ 540)
and you would like to install a newer one:**

**Before you deinstall the old version, please copy the following two directories
and save them somewhere (e.g. on a USB-Stick), so that they can be used
afterwards:**

**C:\Program Files\Microsoft SQL Server\ MSSQL.1
and C:\RAPID\ DATA**

**Then you can deinstall the old one and install the new software. Afterwards,
please replace the newly installed DATA and MSSQL.1 directories by the old
ones.**

Case 2:

**The old version is 380 or 437 and you would like to install a newer one - 437 or
490 (caution: different camera types! Please check, which software is necessary
for you).**

**Please save
C:\RAPID\ DATA and
C:\RAPID\ DATABASE**

And follow the upper instructions.